



## Application Guide

### Abri-Form (All-in-one) – Lying position



1. Fold the brief lengthways with the backsheet turned outwards. Do not touch the inside of the brief.



2. Place the wearer on their side and pass the folded brief from front to back.



3. Fan out the rear end of the brief.



4. Place the wearer on their back. Fan out the front of the brief, creating a channel shape between the legs.



5. Fix the bottom tapes on both sides for a snug, comfortable fit.



6. Fix the top tapes angled downwards

7. Make sure the edges of the brief ease into the groin area and the backsheet is turned away from the skin.

